

Remove the Unit



Transfer into sterile field



Open inner package



Remove contents

Attach the Sensor



Insert battery



Power on



Pair sensor



Attach unit to bracket



Attach sensor to bracket

Calibrate the Unit



Place navigation unit onto the back table in the designated positions and hold steady. Level must be green, but the ball does not need to be in the exact center of the circle.

Audio Feedback:

- Short Beep: Acknowledgement that a registration was accepted.
- Long Beep: Input was not accepted or there was an error.
- Double Beep: Navigation unit completed registration or calibration step successfully.

Assemble Instruments



Insert gear rack



Press levers




Collapse paddles

Attach Electronics and Prepare



Attach unit to bracket

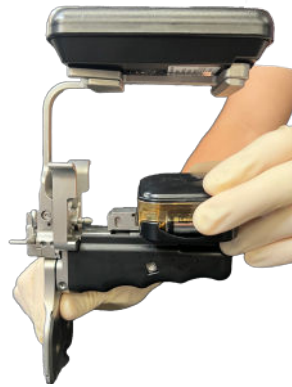


Attach sensor to bracket



Insert calibration bracket

Calibrate

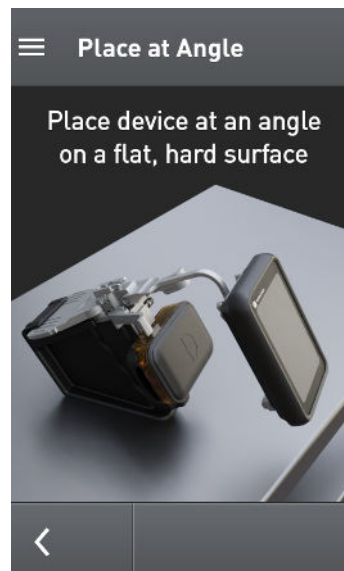

Place the calibration bracket onto the back table in the designated positions and hold steady